

Errors below 100 have to do with reading the data from the flash drive (expiry).

Errors 100-199 have to do with the update file from the therapist.

Errors 100-299 have to do with the resulting updated file on the flash drive.

Process update file errors

- 1 no dongle
 - 2 date error
 - nn as list below
 - 3 buffer allocation error
 - 4 resulting session count invalid
 - 100+nn as list below on update file
 - 100 mismatched dongle number
 - 101 mismatched client id
 - 6 UPDATE= error
 - 7 already used update
 - 8 reallocate buffer error
 - 9 unable to reopen file for write
 - 200+nn error as list below on updated file
-
- 20 not a remote user
 - 22 failed opening EXPIRY
 - 23 memory allocation error
 - 24 CHECK= missing
 - 25 bad CRC
 - 26 DIGEST=missing
 - 27 bad hash
 - 40 CREATED=missing
 - 41 created in future
 - 42 CLIENT= missing
 - 43 client id mismatch
 - 44 WRITTEN= missing
 - 45 written in future
 - 30 wrong dongle number
 - 31 EXPIRATION= missing
 - 32 REMAINING= missing
 - 33 invalid number remaining
 - 34 PLAN= or CLIENT= missing or out of order
 - 35 memory allocation error
 - 36 PROTO= missing
 - 37 LAYOUT= missing